


# Stand-up Support R2D2 – assembly instructions

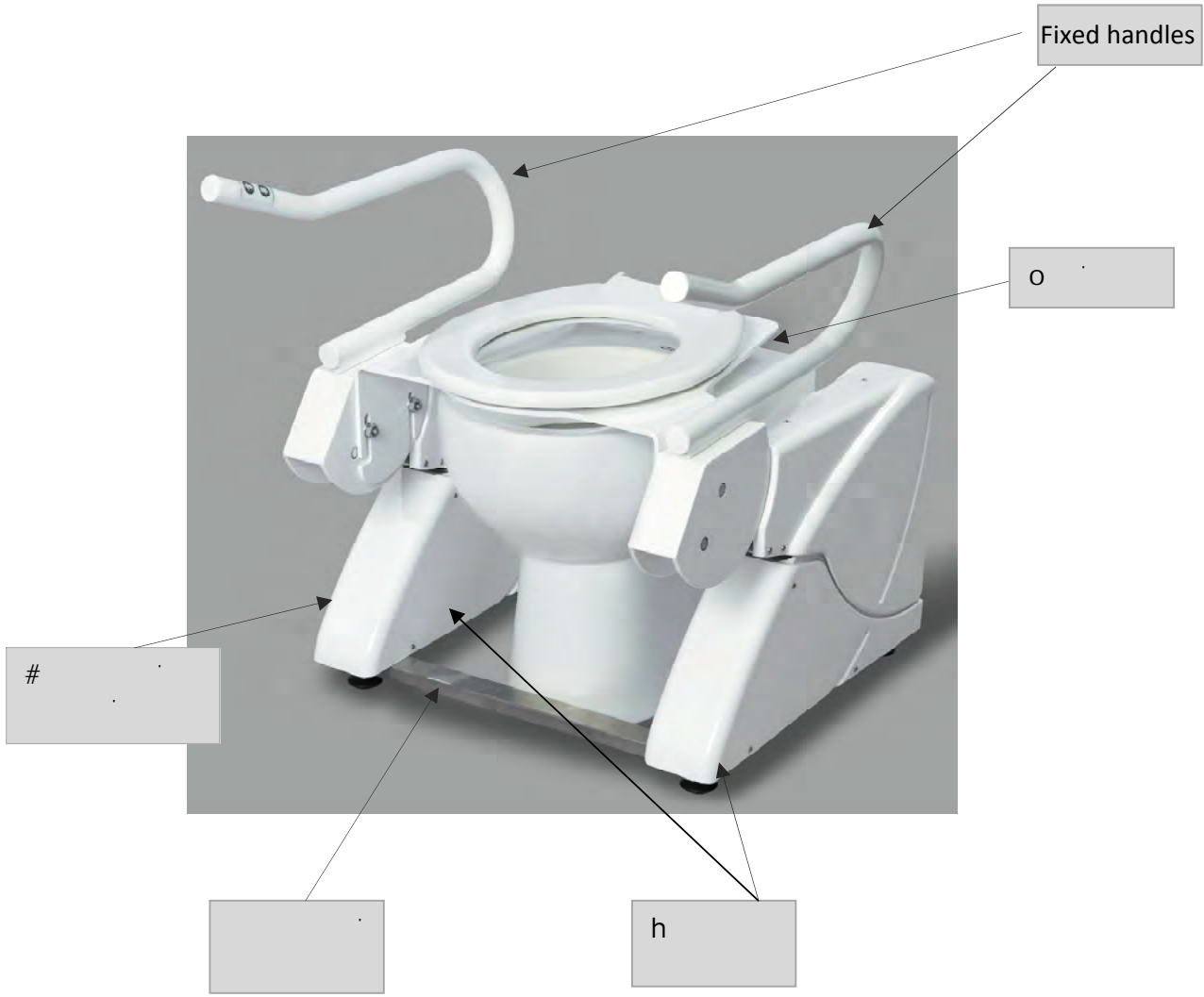

## Available list – Stand-up Support R2D2



### Packaging specifications:

Packaging made of special cardboard

Dimensions: 800 x 700 x 350 mm

Total weight: approx. 41 Kg

---

### Delivery note:

<u>Pos. description</u>	<u>amount</u>
1. R2D2 feet (side panel)	2
2. Connection element	1
3. Fastening set for connection element	1
4. Seat plate	1
5. Fastening set for seat plate	1
6. Set of fuses	1
7. Assembly instruction	1
8. Manual	1

# MONTAGEANLEITUNG

# 1

## Installation of the connection element

### Connection element



### connection alternatives



back




front



1. Take the left and right foot out of the package



2. Choose between the alternative positions and bolt together (gentle) foot and connection element. Connect the cable between left and right foot.

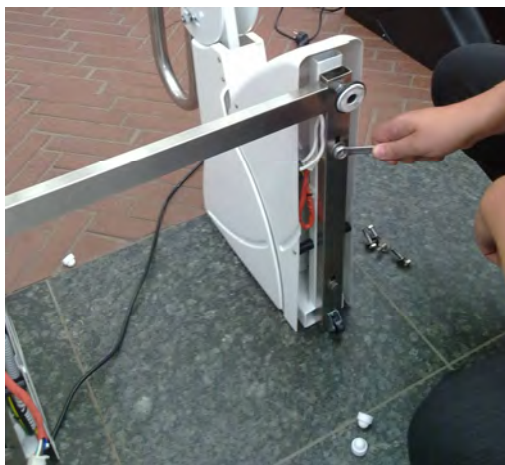

3. After the connection of the cable tight up the screws between the foot and the connection element.

*Hint:* The connection element can be positioned backward and forward by 2 cm.

# 2

## Installation of the seat plate



1. Loosen the screws and nuts.



2. Install the seat plate on the right and left foot and tighten up the nut slightly.



# 3

## Final adjustment

1. Deploy the device on your toilet bowl
2. Set the device on the lowest position!
3. Install the seat plate parallel through the top edge of your toilet bowl  
(distance approx. 3-5cm)
4. Tighten up the 4 screws for fixation between foot stand and seat plate